


## Contra Angle Handpiece/Straight Handpiece

### Spray Lubrication

	<p>Please use the nozzle for E-type, as in the photo.</p> <p>* The spray in the photo is Yoshida Spray.</p>
	<p><b>Spray oil for about 2 seconds.</b> Hold the handpiece tight to avoid being blown away by air-pressure.</p> <p><b>Please confirm that oil oozes out of contra/straight head part.</b></p> <p>Wipe off excessive oil after spraying. Repeat spraying till oil oozing-out becomes clean.</p> <p>* <b>The same method applies to straight handpiece.</b></p>
<p><b>Inside Cleaning</b></p> 	<p>Unscrew the pouch nut and disassemble the head.</p> <p>* Internal cleaning by disassemble the head is more effective to remove blood or chemical inside than spray cleaning.</p>
	<p>Please use the nozzle for chuck cleaning of airturbine, as in the photo.</p> <p>* Don't use chemical or alcohol for inside cleaning. The material can damage internal axis.</p>
	<p><b>Spray oil for about 2 seconds.</b> Hold the handpiece tight to avoid being blown away by air-pressure. Wipe off excessive oil after spraying. Repeat spraying till oil oozing-out becomes clean.</p> <p>After the maintenance, tighten the pouch nut well. Please make trial run before actual use.</p>